

## Saian Ultrasonic Skin Scrubber & FirmAge RF Device Incredible Enzymes Sensitive Skin Protocol

### FEATURES OF TREATMENT:

Incredible Enzymes is a powerhouse against lusterless skin. It optimizes the skin's complexion on the surface and at the cellular level, using a blend of botanical exfoliating enzymes and refinement assistants that work together to rejuvenate and repair tone and texture while brightening and clarifying the skin (pH 5.6).

Saian professional strength ultrasonic skin scrubber used on moist skin on deep cleaning mode exfoliates and thoroughly cleanses the face with ultrasonic vibrations that remove dead skin cells, dirt, and sebum. The skin immediately feels smooth, even, and soft. The second mode uses sonophoresis for an enhanced application of moisturizers, serums, and other anti-aging products.

Saian FirmAge RF and Infrared machine when used with anti-aging serums can firm and tone the face and neck, stimulate collagen production, and visibly reduce the signs of aging. The infrared light activates the cells, promotes blood circulation and metabolism, and keeps skin healthy.

### BENEFITS OF TREATMENT:

- Optimizes complexion on surface and cellular levels.
- Rejuvenates, exfoliates, and clarifies texture.
- Naturally lightens and evens skin tone.
- Deeper exfoliation and detoxifying of the skin.
- Deeper product penetration and effectiveness.
- Improve firmness on the face and neck.
- Visibly reducing signs of aging.

### PRODUCTS:

- Quiet Wash
- Incredible Enzymes
- Mist Me Spritz
- Hush Hydrate Mask
- Relief Bio-Powder
- Soothe Essence Serum or Rare C Serum or Rare Retinal Serum
- Eye Mousse or Brilliant Eye & Lip Serum
- Hush Hydrate Gel, Vital Lipid Lotion or Saffron Meristem Cream
- Broad Spectrum SPF 30

### TOOLS:

- Saian Natural Clinical Professional Strength Ultrasonic Skin Scrubber
- Saian FirmAge Portable Radio Frequency device

### STEPS FOR TREATMENT:

**PRE-CLEANSE:** Apply a dime size amount of **Quiet Wash** directly to skin to remove make up and oils. Then add water to the skin and cleanse. Remove thoroughly with a cool towel or microfiber pads.

**CLEANSE:** Apply a dime size amount of **Quiet Wash** emulsify with tepid water and cleanse the skin. Rinse thoroughly with a cool towel.

**EXFOLIATE:** Apply a thin layer of **Incredible Enzymes** to skin for 3-5 minutes and rinse well with cool towel. (If necessary, perform an additional cleanse using **Quiet Wash**.)

*See Reverse for continued Steps for Treatment, Notes & Contraindications*

**STEPS FOR TREATMENT (CONTINUED):**

**TREAT:** Deeply cleanse and perform extractions with the **Saian Ultrasonic Skin Scrubber** using the protocol from Saian Natural Clinical Skincare. Spray **Mist Me** directly on the skin to keep the skin wet during this treatment.

**MASK:** Apply a quarter size amount of **Hush Hydrate Mask** with 3-5 shakes of **Relief Bio-Powder** in bowl and combine. Liberally apply to the skin with mask brush and let penetrate the skin for 5-10 minutes. Remove residual mask with cool towel.

**TREAT:** Mix 6-10 pumps of desired serums (**Soothe Essence Serum, Rare C Serum** or **Rare Retinal Serum**) in a mask bowl. Apply half the mixture with a mask brush to half of the face and neck. Use **Saian FirmAge RF Device** over the skin using the protocols from Saian Natural Clinical Skincare. Repeat on the other side of the face and neck.

**ENHANCE:** Apply a ¼ pump of **Eye Mousse** to the eye area.

**HYDRATE:** Apply 2-3 pumps of appropriate moisturizer according to skin type (**Hush Hydrate Gel, Vital Lipid Lotion** or **Saffron Meristem Cream**) to face and neck.

**PROTECT:** Apply 1-2 pumps of **Broad Spectrum SPF 30** by patting sunscreen into the skin.

**TREAT:** Apply ¼ pump of **Brilliant Eye & Lip Serum** to the lips.

**NOTES AND RECOMMENDATIONS:**

- Prior to using this product, it is recommended to patch test all clients especially with sensitive and Health-Challenged Skin®.
- Wear gloves and sanitize regularly.
- Use steam & hot towels with extreme caution. Have cool towels available if client is agitated by ingredient activity to remove product quickly.
- Do not heavily rub or try to penetrate the product into the skin.
- Check the client's health history for medications, allergies or sensitivities to any ingredients in this product.
- Do not leave product on skin longer than the recommended time.
- Follow all protocols and instructions for the Ultrasonic Skin Scrubber or FirmAge Radio Frequency listed in the owner's manual provided by Saian Natural Clinical Skincare.

**CAUTIONS AND CONTRAINDICATIONS:**

- Do not use on skin with open lesions, sores or wounds.
- Do not use on broken/cracked, sun-burned or rashy skin.
- Do not use on open comedones, current breakouts or cystic acne (grades 3 & 4).
- Do not use on clients with compromised barriers or active skin conditions.
- Do not use on clients who have allergies to natural fragrances.
- This product contains four enzymes (pineapple, papaya, fig and pomegranate). The use of SPF is mandatory for 2 weeks daily following application of this product.
- It is recommended that use of Vitamin A (retinals and retinols), enzyme masks or scrub/polishing products not be used within 5 days of this treatment.
- Do not wax 3 days before or one week after treatment.
- Check with physician if client is pregnant, breast feeding, on chemotherapy drugs, have a severe autoimmune disease or any health condition that may need a physician's release to perform this treatment.
- Avoid contact with eyes. Keep out of reach of children.
- Please refer to a Physicians Desk reference manual to assure there will be no conflicts with prescribed or OTC medications and ingredients used in this product.
- If any reactions occur during treatment, cease the use of this product immediately.
- Avoid all contraindications for the Ultrasonic Skin Scrubber or FirmAge Radio Frequency listed in the owner's manual provided by Saian Natural Clinical Skincare.

\*Please refer to the product data sheet for Incredible Enzymes for additional questions, concerns and for the complete ingredient listing for this product.

\*Please refer to the owner's manual or pamphlets for all questions regarding the Ultrasonic Skin Scrubber or FirmAge RF device or contact Saian Natural Clinical Skincare with any questions or concerns.