

# Incredible Enzymes Sensitive Skin Protocol

with ZAQ Zayn 2.0 Multifunctional Rejuvenation Device

## FEATURED PRODUCT

### Incredible Enzymes

A high-performance exfoliating treatment formulated for sensitive skin. This advanced enzyme blend combines botanical exfoliants (pineapple, papaya, fig, and pomegranate) with refinement co-factors to gently resurface the skin, promote cellular renewal, and enhance clarity, tone, and texture. pH 5.6.

## FEATURED MODALITY

### Zayn 2.0 Multifunctional Facial Therapy Device

A compact, professional-grade tool integrating four clinically inspired technologies:

- Red & Blue LED Therapy
- Microcurrent Stimulation
- Therapeutic Heat
- Sonic Vibration

Key Benefits:

- Stimulates collagen synthesis
- Reduces acne-causing bacteria
- Enhances facial contouring and muscle tone
- Improves product penetration and skin rejuvenation

## TREATMENT BENEFITS

- Optimizes surface and cellular-level complexion
- Gently exfoliates, clarifies, and refines texture
- Naturally brightens and evens skin tone
- Promotes deeper detoxification and renewal
- Enhances ingredient absorption and efficacy
- Improves firmness in the face and neck
- Visibly reduces signs of aging

## PRODUCTS

- Quiet Wash
- Incredible Enzymes
- Mist Me Spritz or Fresh Breeze
- Hush Hydrate Mask
- Relief Bio-Powder (optional)
- Soothe Essence Serum, Rare C Serum, or Rare Mandelic Serum
- Remedy Rehab Oil
- Berry Boost or Charcoal Mask
- Eye Mousse or Brilliant Eye & Lip Serum
- Hush Hydrate Gel, Vital Lipid Lotion, Saffron Meristem Cream, or HydroMelon Cream
- Broad Spectrum SPF 30

## TOOLS

- ZAQ Zayn 2.0 Device
- **Optional:** Gua Sha tools, mask brush, fan brush, treatment bowls, microfiber pads

## TREATMENT PROTOCOL

### 1. Pre-Cleanse

Apply 1-2 pumps of **Quiet Wash** directly to dry skin. Gently emulsify with water to remove makeup and surface oils. Remove thoroughly with a cool, damp towel or microfiber pad.

*Continued* →

# Incredible Enzymes Sensitive Skin Protocol

## with ZAQ Zayn 2.0 Multifunctional Rejuvenation Device

### TREATMENT PROTOCOL (*continued*)

#### 2. Cleanse

Dispense 1–2 pumps of **Quiet Wash** (or **Clarify Wash** for oily/combination skin). Emulsify with tepid water and cleanse thoroughly. Rinse and remove with a cool towel.

#### 3. Exfoliate

Apply ½ teaspoon or a thin, even layer of **Incredible Enzymes** to the face and neck. Gently massage into the skin and leave on for 3–5 minutes. Rinse thoroughly with cool water and remove with a damp towel. If residue remains, perform a brief secondary cleanse with **Quiet Wash**.

#### 4. Treat with Zayn 2.0

Dispense 6–10 pumps of selected serum (**Soothe Essence**, **Rare C**, or **Rare Mandelic**) into a treatment bowl.

##### Divide Face into 3 Treatment Zones:

1. Left lower face (cheek and jaw line), brow, and center to left side of forehead
2. Right lower face (cheek and jaw line), brow, and center to right side of forehead
3. Neck and Decollete

Apply serum to each zone immediately prior to treating the zone using a fan brush. Perform a 5-minute **Zayn 2.0** session per zone before progressing.

**Avoid the thyroid area during neck treatment.**

Begin at Level 1 for first-time clients; test tolerance before increasing intensity.

#### 5. Mask

Apply 1 teaspoon of **Berry Boost** or **Charcoal Mask** using a mask brush. Allow to penetrate for 7–10 minutes. Remove residue with a cool, damp towel.

#### 6. Massage

Dispense 1 drop of **Remedy Rehab Oil** per zone. Perform gentle facial massage until fully absorbed.

*Optional: Incorporate Gua Sha or facial mushrooms for enhanced lymphatic drainage.*

#### 7. Treat

Spritz skin with **Mist Me** or **Fresh Breeze**. Follow with 2–3 pumps of selected serum across face and neck.

#### 8. Enhance

Apply ¼ pump of **Eye Mousse** to the orbital area using a light patting motion.

#### 9. Hydrate

Apply 2–3 pumps of appropriate moisturizer based on skin type:

- **Hush Hydrate Gel** (± **Relief Bio-Powder** mixed) – lightweight, soothing
- **Vital Lipid Lotion** – barrier repair
- **Saffron Meristem Cream** – anti-aging
- **HydroMelon Cream** – hydration boost

*Optional: Dust Relief Bio-Powder with a brush for calming effect.*

*Continued* →

# Incredible Enzymes Sensitive Skin Protocol

with ZAQ Zayn 2.0 Multifunctional Rejuvenation Device

## TREATMENT PROTOCOL *(continued)*

### 10. Protect

Apply 1–2 pumps of **Broad Spectrum SPF 30**, patting gently into skin.

### 11. Final Treat *(Optional)*

Apply ¼ pump of **Brilliant Eye & Lip Serum** to lips, cheekbones, brow bones, and tip of nose for a luminous finish.

## PROFESSIONAL NOTES & RECOMMENDATIONS

- Patch test all clients 24–48 hours prior, especially those with sensitive or health-compromised skin.
- Wear gloves and sanitize tools between clients.
- Use steam and hot towels with caution; keep cool towels on hand for immediate removal if irritation occurs.
- Avoid aggressive rubbing or over-application.
- Do not exceed recommended dwell times.
- Review client health history for medications, allergies, or sensitivities.
- Follow all **ZAQ Zayn 2.0** guidelines per manufacturer instructions.

## CAUTIONS & CONTRAINDICATIONS

### Do Not Use On:

- Open wounds, sores, or lesions
- Sunburned, cracked, or rash-prone skin
- Active cystic acne (Grade 3–4) or open comedones
- Compromised skin barrier or inflammatory conditions
- Clients allergic to natural fragrances

### Additional Precautions:

- Contains four fruit enzymes — SPF mandatory post-treatment
- Avoid retinoids, enzyme masks, or exfoliants 5 days before/after
- No facial waxing 3 days prior or 1 week post-treatment
- Physician clearance required for pregnancy, chemotherapy, severe autoimmune conditions, or relevant medications
- Avoid eye area contact
- Keep out of reach of children
- Refer to Physician’s Desk Reference for potential drug-ingredient interactions
- Cease treatment immediately if adverse reaction occurs
- Adhere to all **ZAQ Zayn 2.0** contraindications in the official manual

*Protocol Designed for Licensed Estheticians to incorporate a demo of the Zayn 2.0 device into the facial.*

*Pro Tip: Estheticians could incorporate Zayn 2.0 pricing into facial and client purchases the device used during service for home use.*